

City of Philadelphia Public Health Department
500 South Broad Street Philadelphia PA 19146
BinaxNOW™ COVID-19 Ag Card Quiz

Name: _____

Date: _____

Circle T (True) or F (False) for each Question:

- | | | | |
|-----|--|---|---|
| 1. | For testing patient direct nasal swabs, hold Extraction Reagent bottle vertically Hovering 1/2 inch above the TOP HOLE, slowly add 8 DROPS to the TOP HOLE of the swab well. | T | F |
| 2. | The BinaxNOW™ COVID-19 Ag Card device should not be removed from the foil pouch until just before use. | T | F |
| 3. | A nasopharyngeal swab is an acceptable sample type for testing on the BinaxNOW™ COVID-19 Ag Card. | T | F |
| 4. | Direct nasal swab samples may be stored at room temperature for 24 hours. | T | F |
| 5. | It is acceptable to return the nasal swab after patient collection to its original packaging. | T | F |
| 6. | False negative results can occur if the sample swab is not rotated (twirled) prior to closing the card. | T | F |
| 7. | If no lines are seen, or if just the Sample Line is seen, the assay is negative. | T | F |
| 8. | Results should be read promptly at 15 minutes, and not before. Results should not be read after 30 minutes. | T | F |
| 9. | The appearance of a pink-to-purple Control Line and a pink-to-purple Sample Line below it is a positive result. | T | F |
| 10. | Test using only with swabs provided in the kit, and collect direct nasal swabs from individuals suspected of COVID-19 by their healthcare provider within the first seven days of symptom onset. | T | F |